

# NDX Aligners

GUIDE TO CAPTURING PATIENT PHOTOS



FRONT (SMILING)



FRONT (NON-SMILING)



PROFILE



FRONTAL



RIGHT BUCCAL



LEFT BUCCAL



UPPER OCCLUSAL



LOWER OCCLUSAL

## EXTRAORAL PHOTOS

- Use a dark background to reduce any halo effect from a flash.
- Position the camera at the patient's nose level.
- Include the entire chin and top of the head in the frame.
- Capture the right side for the profile picture.
- Ask patients with long hair to tuck it behind their ears.



## INTRAORAL PHOTOS

- Use a cheek retractor for the patient to assist in holding the mouth in place.
- Start with a frontal shot while the patient sits knee-to-knee with the photographer.
- For right/left buccal shots, have the patient pull the retractor on one side while keeping the opposite cheek relaxed to ensure all teeth are visible.



FRONTAL




RIGHT  
BUCCAL



LEFT  
BUCCAL

## OCCLUSAL PHOTOS

- Use a Lip Columbia retractor and a mirror.
- Ask the patient to lift their head high and open wide.
- The mirror should rest flat against the opposing arch teeth.
- Ensure all teeth are visible in the mirror.
- Keep the nose out of the frame.
- Use an anti-fog mirror for clarity.

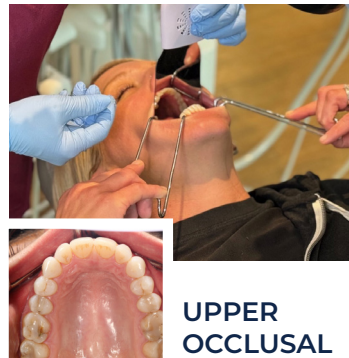

UPPER  
OCCLUSAL



LOWER  
OCCLUSAL

**Contact your local sales representative for assistance or for more information, please visit [nationaldentex.com/aligners](https://nationaldentex.com/aligners).**